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USER MANUAL

ORTEZA KOLANA KNEE ORTHOSIS

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EN **Reh4Mat** products are appreciated and recommended by specialists (doctors, physiotherapists) and users. Our products are produced in Poland and made of high quality materials. In case of any questions, feel free to contact us: export@reh4mat.com.

INDICATION

Knee orthosis stabilizes and relieves pressure of the knee joint. Relieves the pain and improves the healing. The support is used in case of rheumatic, orthopedic and neurological disorders, as an treatment after surgery and in physical therapy.

CARE AND MAINTENANCE

- Product should be maintain in cleanness and use accosing the manufacturer recommendations.
- Product should be maintain in cleanness and use accosing the manufacturer recommendations. Clean the product systematically, at least once a month or more of ten if necessary, in accordance with the manufacturer's recommendations, ensuring that it is kept in good condition.
- If the product holds lateral splints or shells and it is possible to dismantle them without any tools, remove them before washing.
- Wash by hands in lukewarm water with soap at 30°C.
- Sluice carefully, use when completely dry.
- Do not iron, do not chlorinate, do not dry clean, do not tumble dry.
- Keep it in a dry and clean zone.
- Protect from moisture, extreme temperatures, sunlight, dry away from sources of warmth.

CONTRAINDICATIONS

- The product should be used according to the recommendations of a physician or physiotherapist.
- Do not use the product in the wrong size. Only a correctly fitted size guarantees effectiveness and safety.
- Do not use the product by people and children with mental disorders without guardian control.
- Do not use the product on patients with superficial and deep sensation disorders without the supervision of a guardian.
- The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occurring, it is necessary to contact a physician.
- In case of varicose veins, Deep Vein Thrombosis (DVT), oedema the using of orthosis is allowed only after contact with physician.
- Do not use the product against its purpose of use.
- Do not use the product in case of damage or shape's deformation.
- Do not attempt to service this product yourself. Independent interference with the design of the product may result in permanent damage to the user's health.
- Do not use the accessories which are not recommended by manufacturer.
- Do not use the product near an open flame or on a hot surface. This can seriously injure the user and lead to serious damage to health.
- Use the orthosis during night should by done only if this was prescribed by your doctor.

SAFETY RULES

- The product should be used according the manual's recommendations. If you have any questions, please consult your car eprofessional.
- Before using the product, read this manual carefully. Following the instructions included in the manual will provide product's safety an effectiveness. If you have any questions, please consult your care professional.
- Using of the product not according to the manual may cause serious complications.
- Please, keep the manual for future.
- Check the product's condition. If the product is damaged, discontinue use and contact the manufacturer. Only an undamaged product and proper us eguarantee its right and safe working.
- Manufacturer is responsible only for hidden defects in the product or mechanical damages occurred in the natural way of using.
- Manufacturer is responsible for product's effectiveness and safety only in case of purchasing the product in the authorized seller.
- Pain is always the sign of warning. If the pain maintains or increases, it is advisable to consult your doctor or physical therapist.
- For single patient use only.
- The product should be used according to the recommendations of a physician or physiotherapist.
- Before first using, check the size of the product. Only good size guarantees safety and effectiveness. A wrong size of the product decreases effect of treatment, increases discomfort and wrong position of the user. Proper use of the brace diagnosis by physician orr physiotherapist.
- Control using the product according to the recommendations.
- Control the skin condition and in case of skin lesions occurring, it is necessary to contact a physician.
- In case of varicose veins, Deep Vein Thrombosis (DVT), oedema the using of product is allowed only after contact with physician.
- Before using remove package and accessories like papers, cards or labels.

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The manufacturer holds the ISO 13485 international Quality Management System certificate.

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